


Eurovema

Creates mobility and ergonomics.


Exploded views/Spare part

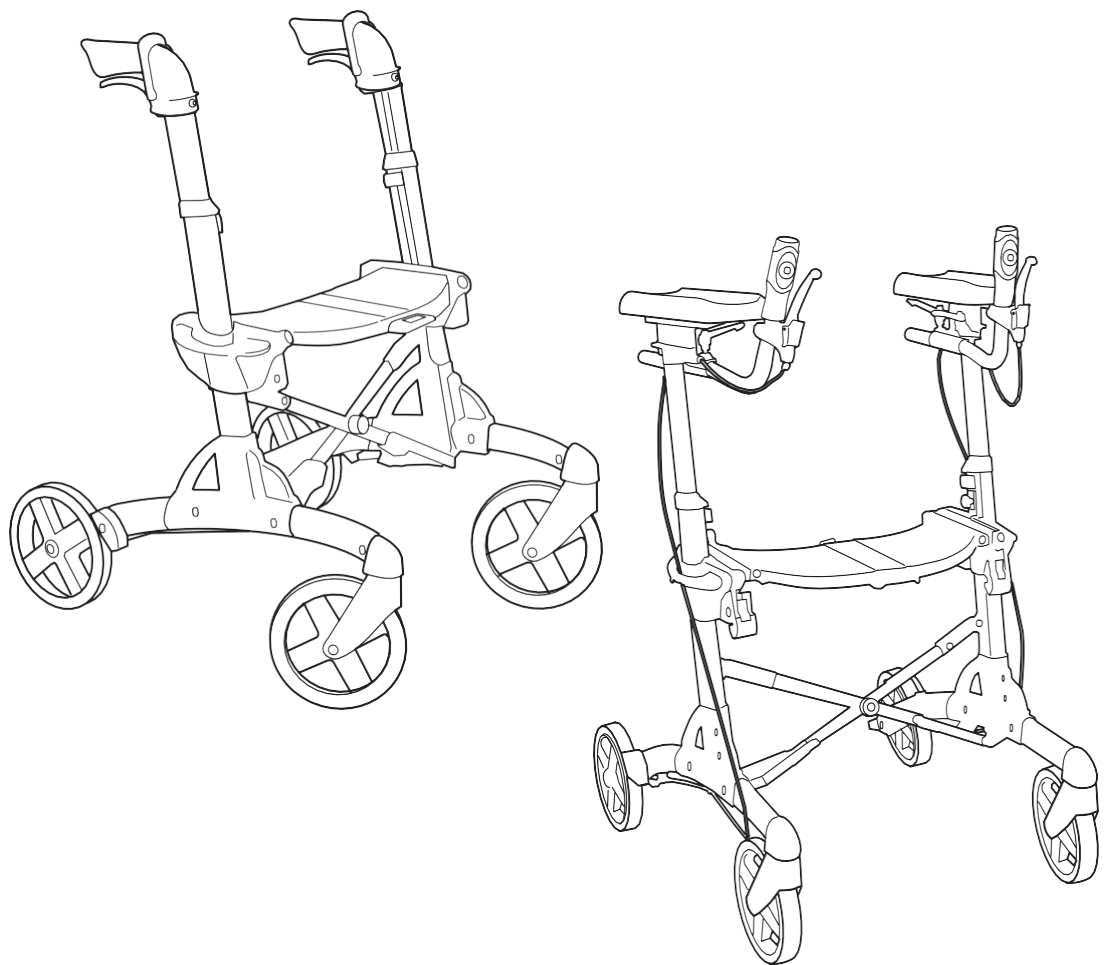


VOLARIS

Compact, Kid, Smart,
Discovery, Discovery Wide,
Patrol, Patrol Wide, RA


Compct/Kid

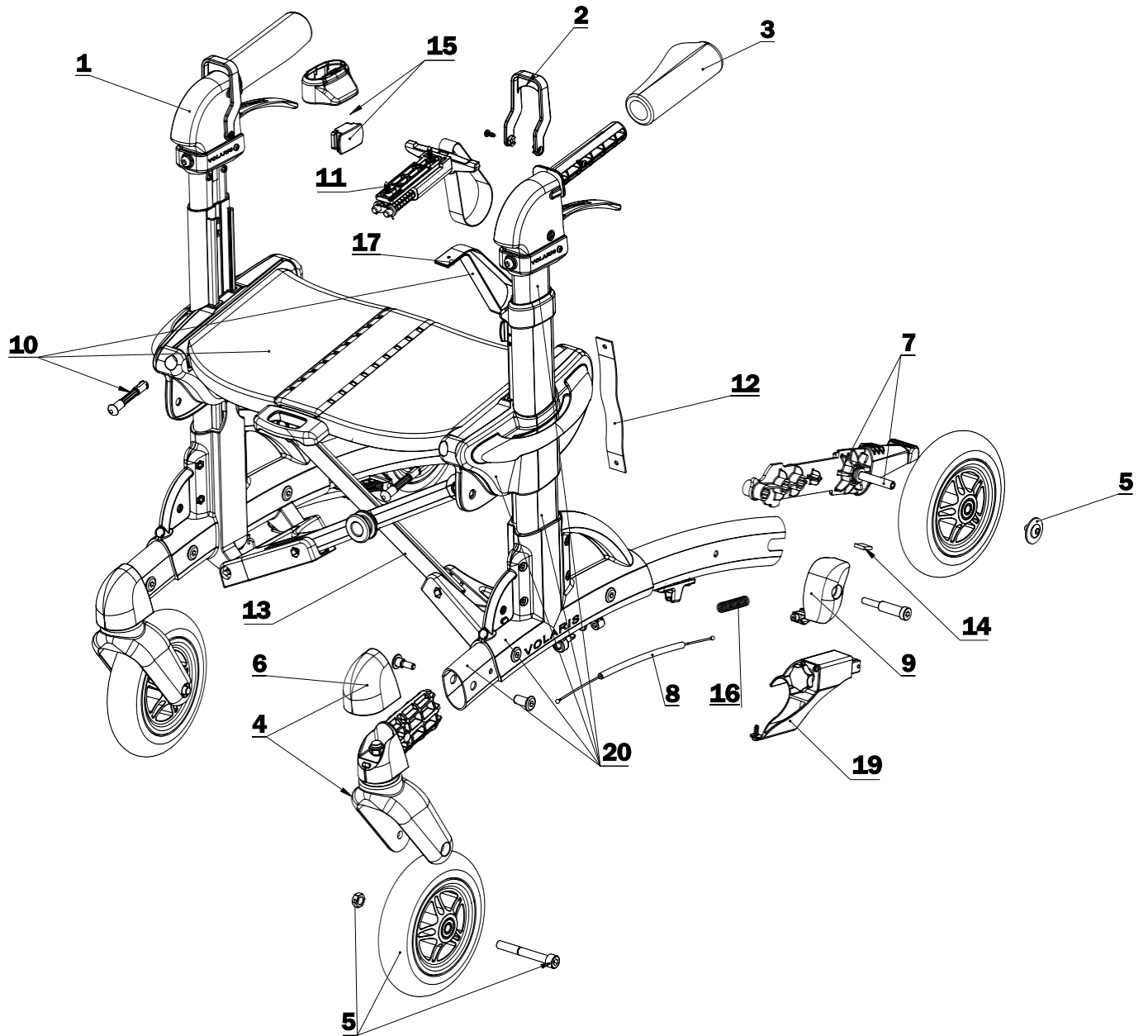


Smart/Discovery/Patrol

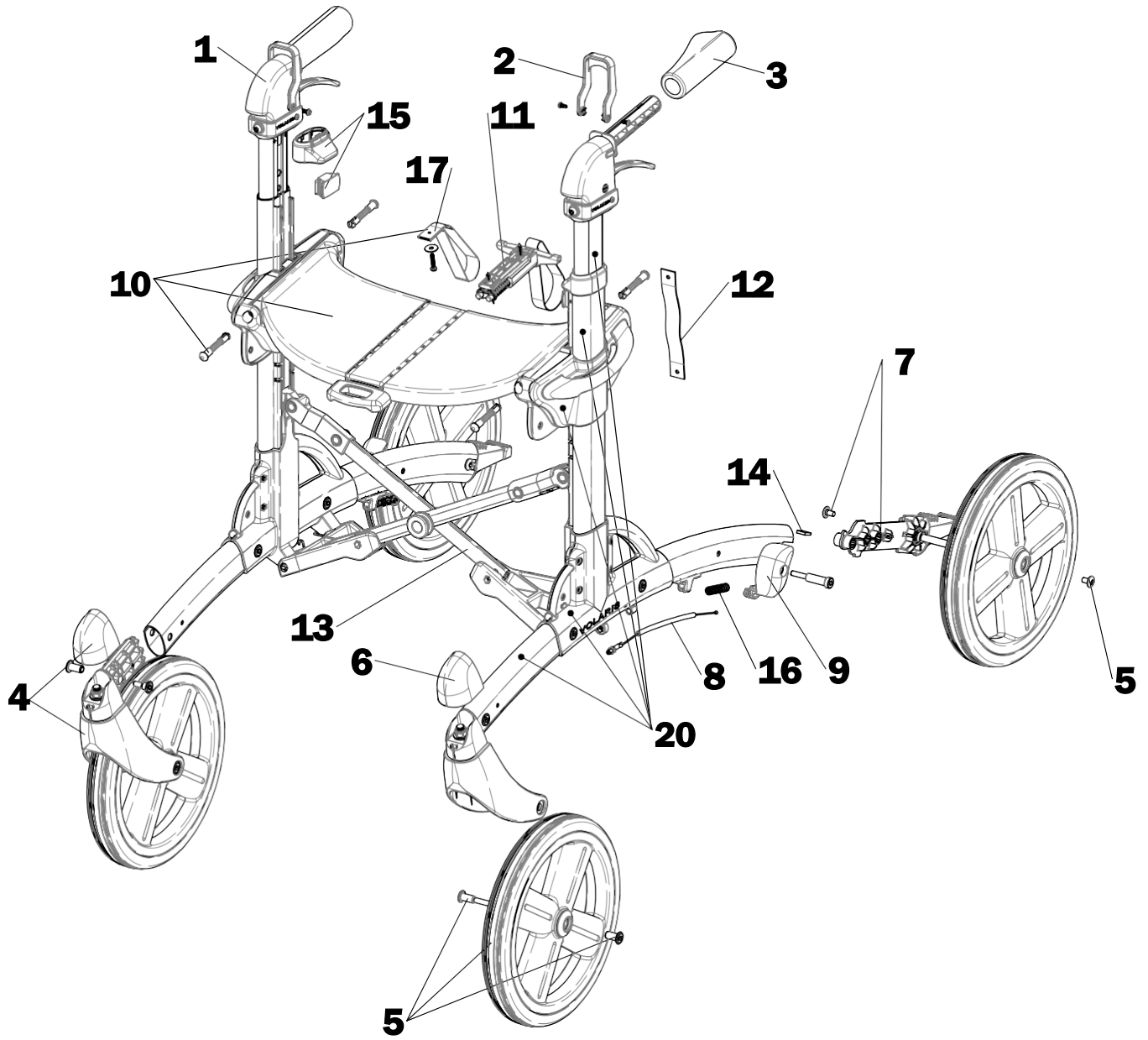

Discovery Wide/Patrol Wide

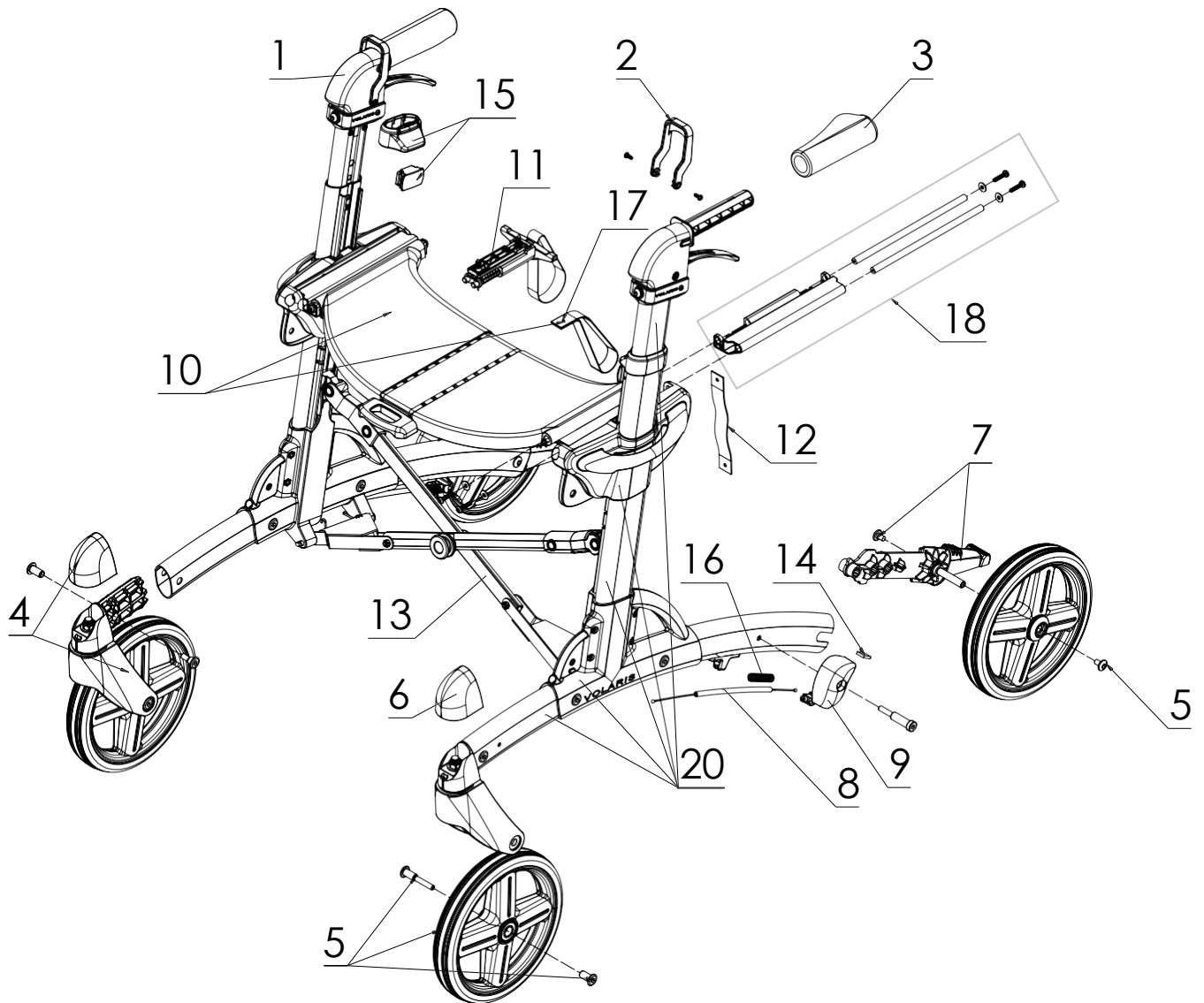

RA/RA låg/low

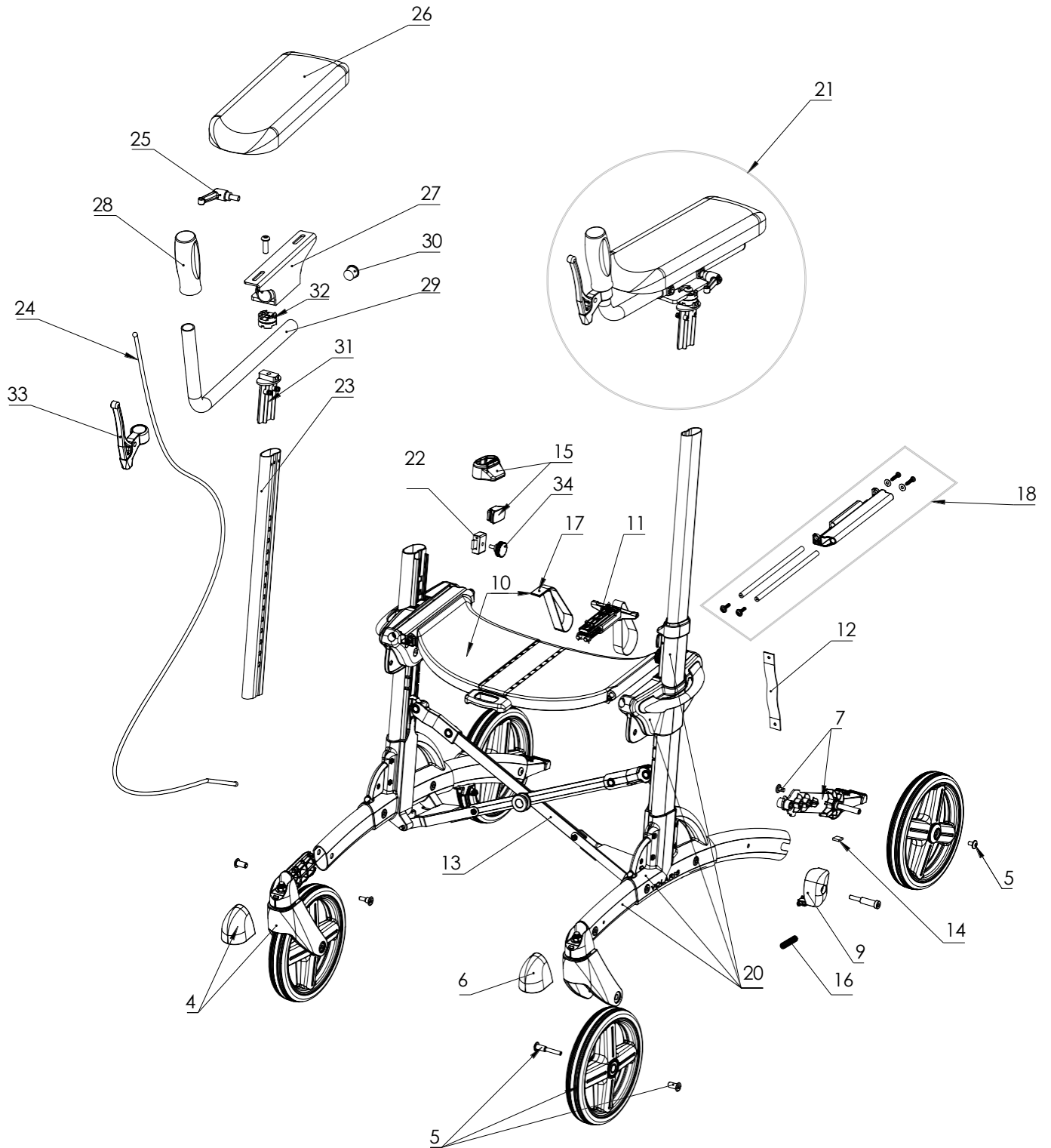

Exploded views/spare part Compct, Kid, Smart, Discovery, Patrol, Discovery Wide, Patrol Wide, RA incl. low

Article no.	Title:
1	4017942060 Handle orange cpl L
	4017942070 Handle orange cpl R
	4017942062 Handle Low orange cpl L
	4017942072 Handle Low orange cpl R
	4017942061 Handle black cpl L
	4017942071 Handle black cpl R
	4017942063 Handle Low black cpl L
	4017942073 Handle Low black cpl R
	4017942062 Handle Low orange cpl L
	4017942072 Handle Low orange cpl R
	4017942063 Handle Low black cpl L
	4017942073 Handle Low black cpl R
2	4017941950 Parking brake device orange
	4017941951 Parking brake device black
3	4017940225 Handgrip L/R, pair
	4017940226 Handgrip Left
	4017942400 Handle Kid cpl R
	4017942410 Handle Kid cpl R
	4017940225 Handgrip SMART L/R, pair
	4017940226 Handgrip SMART Left
	4017930311 Handgripp Kid green
4	4017941080 Front wheel cpl S7
	4017942180 Front wheel cpl Discovery
	4017941870 Front wheel cpl PATROL
	4017942390 Fork compact/ kid
5	4017941003 Wheel S7 incl screws
	4017942161 Wheel Discovery incl screws
	4017941851 Wheel PATROL incl screws
	4017940304 Rear wheel Compact Kid incl screws
6	4017941050 Cover fork S7 grey
7	4017941020 Rear wheel bracket S7 cpl L
	4017941021 Rear wheel bracket S7 cpl R
	4017941024 Rear wheel bracket Discovery cpl L
	4017941025 Rear wheel bracket Discovery cpl R
	4017941022 Rear wheel bracket PATROL cpl L
	4017941023 Rear wheel bracket PATROL cpl R
	4017941026 Rear wheel bracket Compact Kid L
	4017941027 Rear wheel bracket Compact Kid R
8	4017940210 Wire SMART incl nut
9	4017941570 Brake yoke S7 L
	4017941580 Brake yoke S7 R
	4017941571 Brake yoke Patrol Discovery L
	4017941581 Brake yoke Patrol Discovery R
10	4017941280 Seat cpl orange with strap and pins
	4017941282 Seat cpl black with strap and pins
	4017942280 Seat cpl Compact orange
	4017942281 Seat cpl Compact Kid black
11	4017942550 Folding lock orange
	4017942551 Folding lock black
12	4017940202 Transport strap S7 incl screw
13	4017941390 Spare part Cross frame Shadow line
	4017941391 Spare part Cross frame Shadow line
	4017941393 Spare part Cross frame Wide compl.
	4017941394 Spare part Cross frame Compact cpl.
14	4017940032 Brake metal S7 cpl
15	4017941352 Handle adj.knob S7 orange pair
	4017941353 Handle adj.knob S7 black pair
16	4017930037 Brake spring S2 S3
17	4017940161 Strap S7 orange
	4017940242 Strap S7 black
18	4017940265 Spacer Wide seat
19	4017911780 Slow down brake
20	4017942600 Side SMART L
	4017942610 Side SMART R
	4017942620 Side SMART Low L
	4017942630 Side SMART Low R
	4017942640 Side Patrol L
	4017942650 Side Patrol R
	4017942660 Side Patrol Low L
	4017942670 Side Patrol Low R
	4017942680 Side Compact L
	4017942690 Side Compact R
	4017912600 Chassie RA
21	2400350H Underarm support RA cpl R
21	2400350V Underarm support RA cpl L
22	S7-135 Lock heightadj RA
23	S7-138 Handleprofile RA short
23	S7-139 Handleprofile RA long
24	S7-141 Wire RA
25	7WN500-8*16 One hand lever wn500 65-m8x16
26	400350 Underarm plate RA
27	S7-112 Plate attagement RA
28	400360 Handgrip RA
29	S7-119 Handlepipe RA
30	7IKP-22*19 Plastic plug IKP-2218-20S
31	S7-118 Profile attachment RA
32	2400340 Joint Underarm support RA
33	760001-1200 Brakehandle
34	7RATT-M5*10 RA Knob R3010 M5*10 S



Eurovema

Creates mobility and ergonomics.

Baldersvägen 38 | SE-332 35 Gislaved | Sweden

Tel. +46 371 390 100 | www.eurovema.se | info@eurovema.se